AbilEx*

Oral Motor Exerciser
Gymnastique Buccale
Ejercitador Bucal

TrudellMed.com/AbilEx

Trudell Medical International

EN ABOUT **AbilEx*** ORAL MOTOR EXERCISER (OME)

1. Designed to safely stimulate and exercise parts of the mouth.
2. Intended to maintain sensation, range of motion, and co-ordination of the tongue.
3. Helps develop strength of the lips, tongue, jaw, and mouth.
4. Made of soft, comfortable silicone rubber making it easy to grip.
5. This device is intended for single patient use only.

CARING FOR YOUR **AbilEx*** OME

AbilEx* OME is packaged in a convenient storage case to maintain cleanliness between uses.

After each use or if the device becomes dirty:

1. Gently wash the **AbilEx*** OME with mild soap under running water.
2. Dry on a clean surface and return to the storage case.
3. Replace after 30 days of daily use or sooner if holes, tears or punctures are visible on inspection.

NOTE: Do not clean the **AbilEx*** OME in a dishwasher, boiling water, or steam sterilizer.

Do not clean with strong chemicals (example: bleach or peroxide solutions).

BEFORE YOU BEGIN

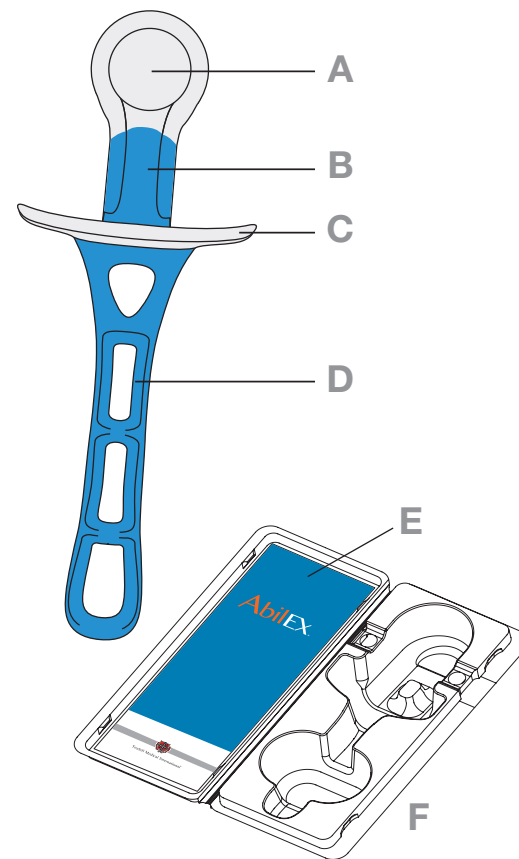
- Read this entire **AbilEx*** OME user guide
- Brush teeth or dentures and rinse mouth
- If you have a full upper denture, remove it
- Wash your hands
- Be seated or resting in an upright position

⚠ WARNING: Do NOT use **AbilEx*** OME with:

- dental disease, broken teeth
- sharp or damaged dental appliances
- severe cognitive impairment
- tendencies towards aggressive or reflexive biting
- open sores on lips, tongue or inside of mouth

DO NOT actively chew on the **AbilEx* OME as this may damage the device and pose a potential choking hazard.**

- Consult your healthcare provider if you have a history of TMJ dysfunction
- Inspect the **AbilEx*** OME and do not use if there are holes, tears or punctures
- Lightly rinse the **AbilEx*** OME under tap water prior to each use
- The **AbilEx*** OME should only be used in the mouth and should not be inserted beyond the shield (Component C)
- Treatment should begin gradually and increase as tolerated
- Work towards 3 sets of suggested exercises 3 times per day, or as directed by your health care provider

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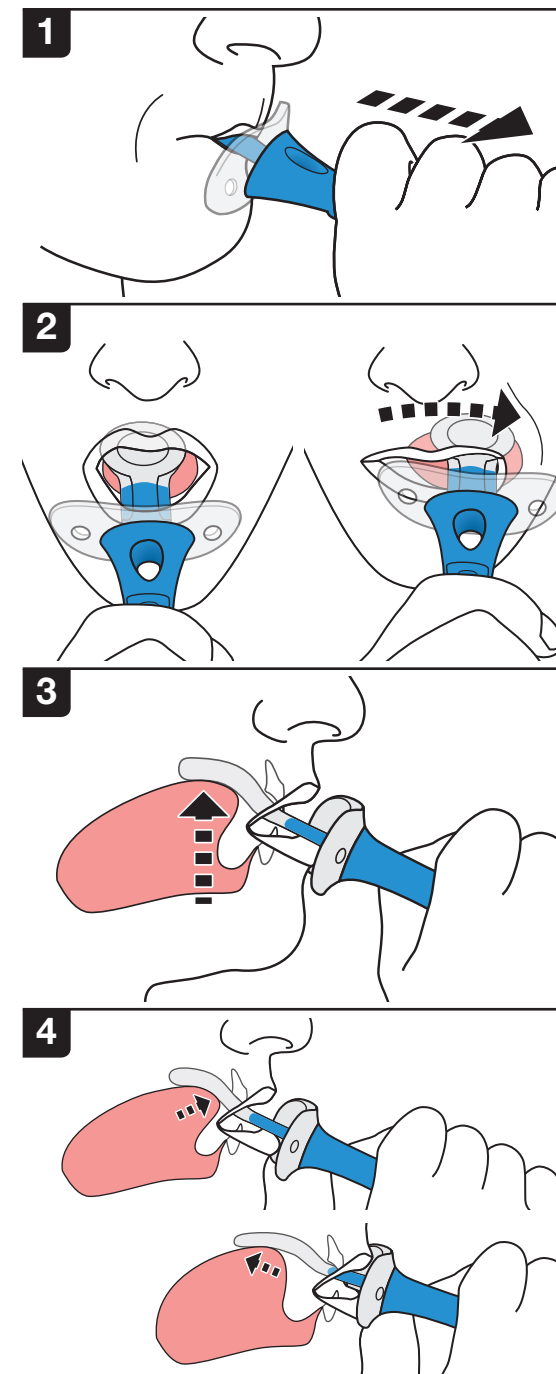
SUGGESTED EXERCISES

These exercises are provided for information purposes only. Consultation with a Speech Language Therapist or healthcare provider is recommended prior to use.

Do NOT actively chew on the **AbilEx*** OME while performing the suggested exercises.

1 Lip Press and Pull (Image 1)

- Place the **AbilEx*** OME in your mouth with the bulb flat on the center of your tongue
- Tightly press your lips around the tether of the **AbilEx*** OME
- Keep the bulb inside your mouth by pressing your lips firmly together
- Gently pull on the **AbilEx*** OME handle for 3-5 seconds
- Relax and repeat 5 times

2 Bulb Side to Side (Image 2)

- Place the **AbilEx*** OME in your mouth with the bulb flat on the center of your tongue
- Move the bulb from the center to the left side of your mouth with your tongue and hold for 3-5 seconds
- Relax and repeat 5 times
- Perform the same action to the right side of your mouth with your tongue
- Relax and repeat 5 times

3 Tongue Push Up (Image 3)

- Place the **AbilEx*** OME in your mouth with the bulb flat on the center of your tongue
- Push up with your tongue to press the **AbilEx*** OME against the roof of your mouth
- Hold for 3-5 seconds
- Relax and repeat 5 times

4 Bulb Front to Back (Image 4)

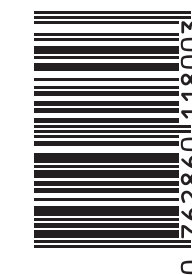
- Place the **AbilEx*** OME in your mouth with the bulb flat on the center of your tongue
- Push the bulb against the roof of your mouth with your tongue
- Creating a slight suction, move the bulb forward and backwards along the roof of your mouth 5 times using your tongue
- Relax and repeat 5 times

NOTE: If redness or soreness occurs in the tissue of the mouth, tongue or lips, stop use and contact your healthcare provider.

Manufactured by:
Trudell Medical International
725 Third Street
London, Ontario, Canada, N5V 5G4
1-800-465-3296 in North America
1-519-455-7060 outside North America
customerservice@trudellmed.com
www.trudellmed.com

Quantity: 1
REF 118501

118004-001 Rev F. *trade marks and registered trade marks of Trudell Medical International. Copyright © Trudell Medical International 2015, 2016, 2017. All rights reserved.

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