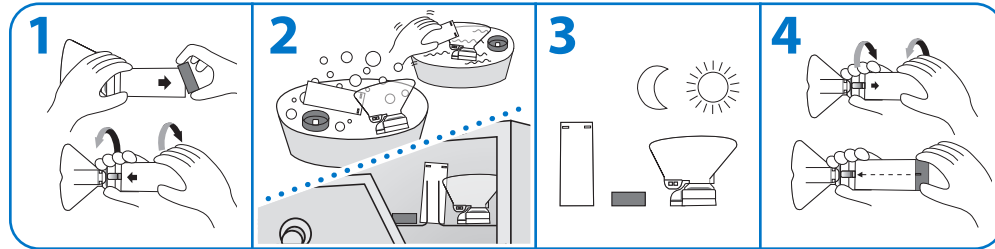
## CLEANING INSTRUCTIONS FOR THE CHAMBER

This chamber can be used right out-of-package and then cleaned weekly.

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- 1 Remove the backpiece. To detach the frontpiece, twist chamber as shown.
- 2 Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently. Rinse parts in clean water. **OR** Place parts in top rack of dishwasher. Ensure product is securely placed face up. Run the dishwasher on a normal or light cycle. Do not heat dry.
- 3 Shake out excess water and allow to air dry in a vertical position. Ensure parts are dry before reassembly.
- 4 To reassemble, fit the frontpiece on the end of the chamber and twist firmly until securely locked into position. Center the alignment feature on the backpiece with the **Flow-Vu®** Inhalation Indicator, as shown. Press firmly to attach the backpiece.

### Notes

- Product should be replaced after 12 months of use.
- Do not boil or sterilize.
- This product contains no latex.
- Do not share this medical device.
- Do not remove the whistle from the backpiece.
- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Dishwashing with overly dirty dishes is not recommended.
- If cleaning in a dishwasher use a rinse aid.
- For cleaning of the inhaler, follow the instructions supplied with the inhaler
- Put cap back on the mouthpiece and store in a clean, dry area when not in use.
- Testing has been conducted with the following disinfectant solutions:
  - Milton®; Dodie®; Perasafe®
  - Test results indicate that the solutions tested have no adverse effect on the performance of the device.

### Cautions

- Product may be permanently damaged if boiled, sterilized or cleaned in a dishwasher at a temperature above 70°C.

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# AeroChamber Plus Flow-Vu<sup>®/MD</sup>

Anti-Static Valved Holding Chamber

Manufactured by / Fabriqué par  
**Trudell Medical International**  
 725 Baransway Drive  
 London, Ontario, Canada N5V 5G4  
 ☎ +1-519-455-7060 📠 +1-519-455-6478  
 info@trudellmed.com [www.trudellmed.com](http://www.trudellmed.com)

Designed and Developed in Canada  
 Conçu et développé au Canada

103077-001 Rev B

**TRUDELL MEDICAL INTERNATIONAL**

## INTENDED USE

This chamber is intended to be used along with a metered dose inhaler or soft mist inhaler to deliver aerosol medication to your lungs as prescribed by your healthcare provider. If you have questions about the performance or use of this product, please contact your healthcare provider.

## BEFORE EACH USE

This chamber can be used directly out-of-package. Ensure these instructions and the instructions supplied with the inhaler have been read and are kept available at all times. Carefully look into the chamber. If there is any visible dust or other debris inside, then clean the chamber. Replace immediately if it is damaged or has missing parts.

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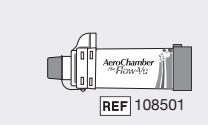
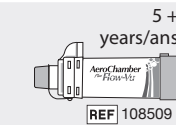
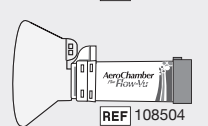
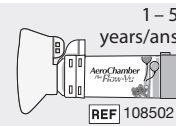
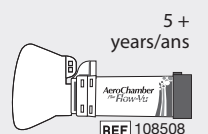
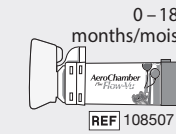
BREATHE  
 the lung association™

The Lung Association believes that using the **AeroChamber Plus<sup>®</sup> Flow-Vu<sup>®</sup>** VHC with your metered dose inhaler (puffer) helps to ensure medication is delivered to where it is needed in your lungs.

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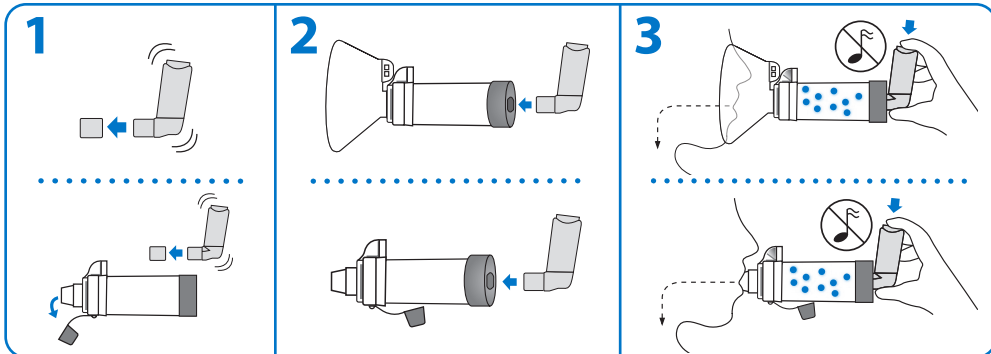
Single patient - multiple use  
 Monopatient - utilisation multiple

**LATEX**

## HOW TO USE YOUR CHAMBER WITH METERED DOSE INHALERS (PUFFERS)

For preparation of the inhaler, follow the instructions supplied with the inhaler.

**1** Remove caps from the inhaler and chamber. Shake the inhaler immediately before use as per the instructions supplied with it.

**2 MASK:** Insert the inhaler into the backpiece of the chamber. Apply mask to face and ensure an effective seal.

**MOUTHPIECE:** Insert the inhaler into the backpiece of the chamber. Put mouthpiece into mouth and close your lips around it to ensure an effective seal.

**3 MASK:** Exhale then press the inhaler once at the beginning of a slow inhalation. Hold mask in place and breathe in and out through the chamber for 5-6 breaths.

**MOUTHPIECE:** Exhale then press the inhaler once at the beginning of a slow inhalation. Inhale slowly and deeply through the chamber until a full breath has been taken. Hold your breath for 5-10 seconds before exhaling. **OR** Exhale and press the inhaler once at the beginning of a slow inhalation. Breathe in and out through the chamber for 2-3 breaths keeping lips sealed around chamber mouthpiece.

### ⚠ Cautions

- Do not use with Dry Powder Inhalers or Nebulizers. This chamber has not been designed to deliver these medications so you may not get the proper dose.
- Failure to follow instructions may affect medication delivery or cause inhalation issues.

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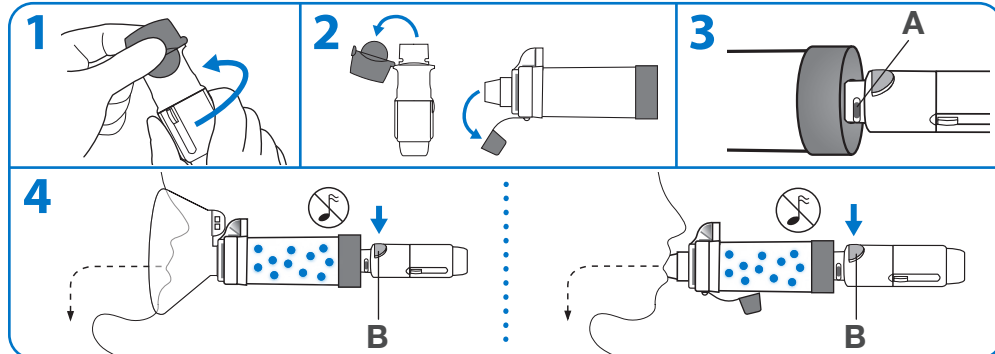
## HOW TO USE YOUR CHAMBER WITH RESPIMAT<sup>+</sup> SOFT MIST INHALERS

For preparation of the inhaler, follow the instructions supplied with the inhaler.

**1** Hold the inhaler upright and turn the clear base in the direction of the arrows on the label until it clicks. (Keep the inhaler cap closed).

**2** Open the cap of the inhaler until it snaps fully open. Remove the cap from the chamber.

**3** Insert inhaler into the backpiece of the chamber. **Do not cover the air vents (A)** on the inhaler.

**MASK:** Apply mask to face and ensure an effective seal.

**MOUTHPIECE:** Place mouthpiece into mouth and close lips around it to ensure an effective seal.

**4 MASK:** Breathe out gently and press the **dose release button (B)** on the inhaler at the beginning of a slow inhalation. Maintain seal for 5 – 6 breaths after the inhaler button is pressed.

**MOUTHPIECE:** Exhale then press the **dose release button (B)** on the inhaler at the beginning of inhalation. Inhale slowly and deeply through the chamber until a full breath has been taken. Hold your breath for 5-10 seconds before exhaling. **OR** Exhale then press the **dose release button (B)** on the inhaler at the beginning of a slow inhalation. Breathe in and out through the chamber for 2-3 breaths keeping lips sealed around chamber mouthpiece.

### ⚠ Warning

- Do not leave the device unattended with children. This is not a toy.

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### 📌 Notes

- Inhalation — SLOW DOWN if you hear the whistle sound, it means you are inhaling too quickly (on adult models).
- If someone is helping you they can use the **Flow-Vu<sup>®</sup>** Inhalation Indicator to ensure a good seal, coordinate pressing the inhaler with inhalation and to count the number of breaths taken. The **Flow-Vu<sup>®</sup>** Indicator moves toward you as you inhale and only moves if you have a proper seal.
- Administer one (1) puff at a time. Follow instructions supplied with the inhaler on how long to wait before repeating.

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