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# Aerobika\*

Oscillating Positive  
Expiratory Pressure Device  
with Manometer Accessory

**TRUDELL MEDICAL**  
INTERNATIONAL

**Communauté**

# *my***AERO**<sup>\*</sup> Community

## **Join the myAERO<sup>\*</sup> Community**

- Content to help you get started and get the most from your device
- Tips to manage your respiratory symptoms
- Opportunities to participate in contests and special offers
- Annual reminders to replace your device

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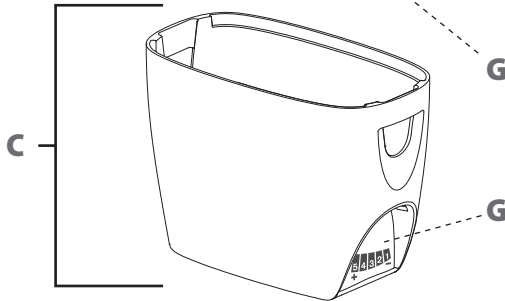
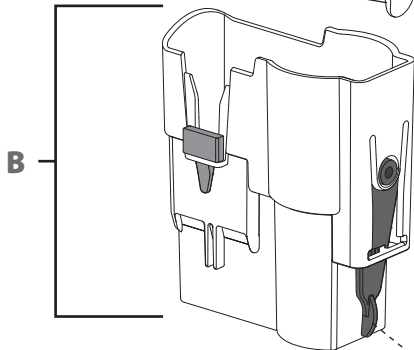
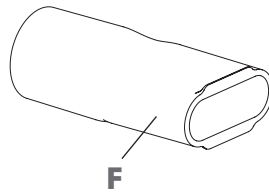
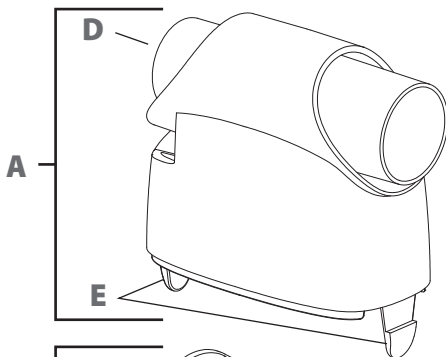
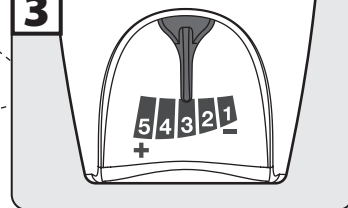
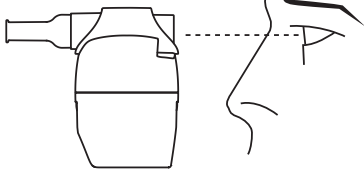
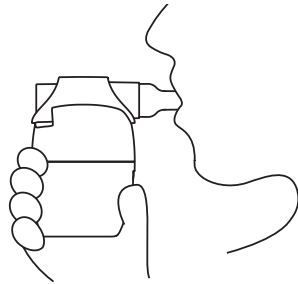
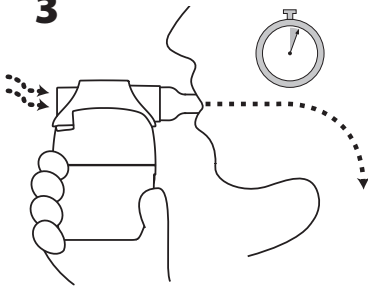
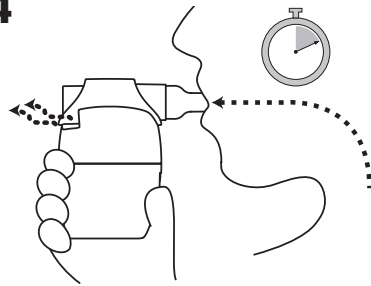
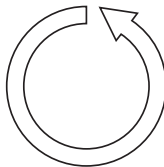
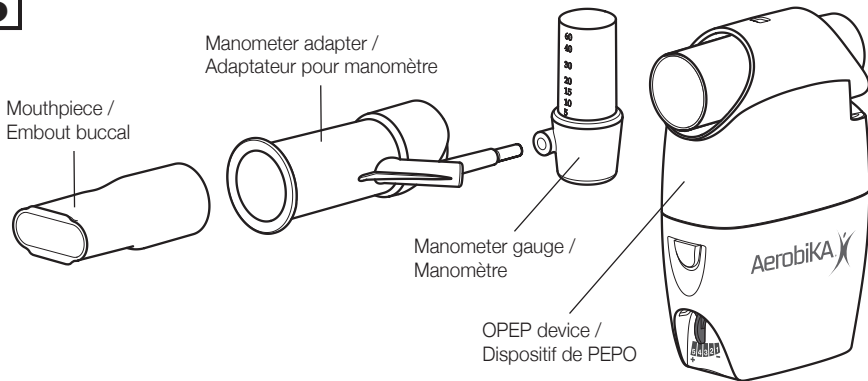
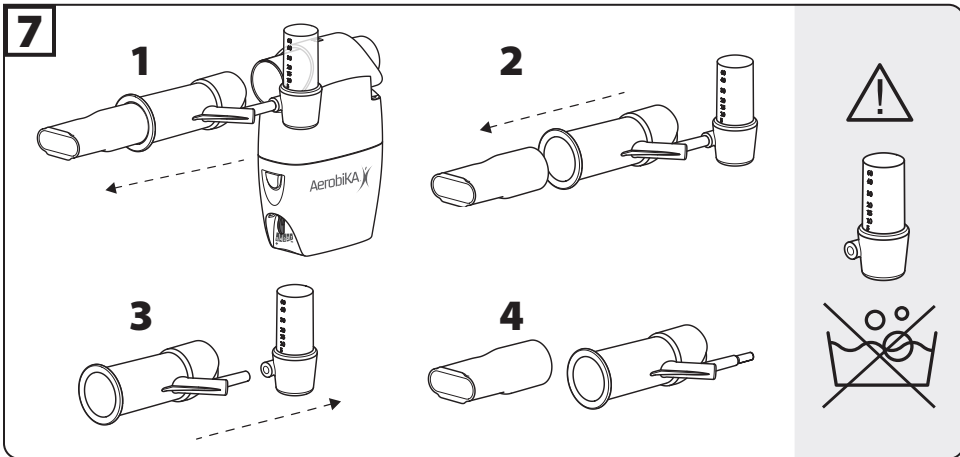
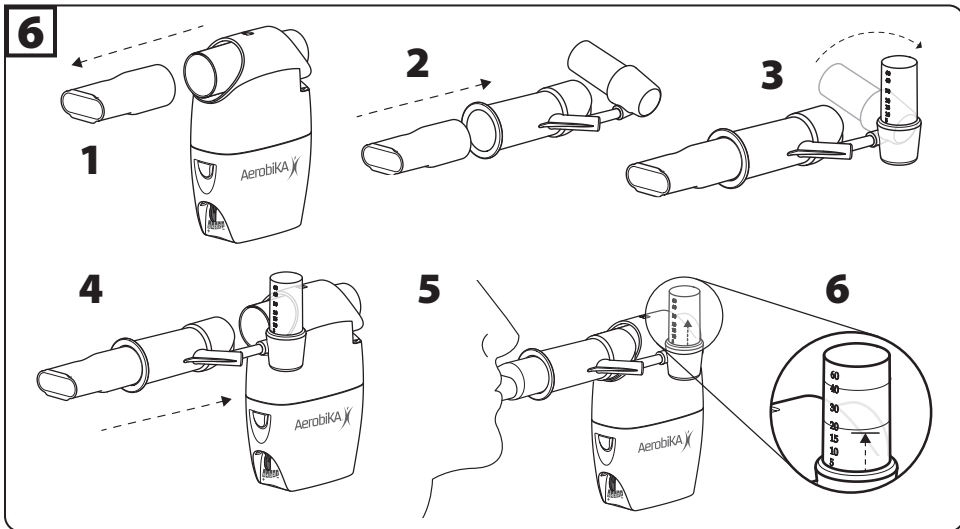
### Aerobika\* Device

#### INTENDED USE

The **Aerobika\*** Oscillating Positive Expiratory Pressure device is intended for use as a Positive Expiratory Pressure (PEP) device. The **Aerobika\*** device may also be used simultaneously with nebulized aerosol drug delivery. The device is intended to be used by patients capable of generating exhalation flow of 10 L/min for 3–4 seconds.

#### INDICATION(S)

The **Aerobika\*** Oscillating PEP device is an airway clearance therapy device used to mobilize excess lung secretions and improve breathing. It can be used to manage respiratory conditions such as COPD, Bronchiectasis and Cystic Fibrosis. Single patient use only.

#### CONTRAINDICATIONS

Use of the device is not recommended for the following conditions:

- Patients unable to tolerate the increased work of breathing

- Intracranial pressure (ICP) > 20 mm Hg
- Hemodynamic instability (blood pressure instability)
- Recent facial, oral, or skull surgery or trauma
- Acute sinusitis
- Epistaxis (bleeding nose)
- Esophageal surgery
- Nausea
- Active hemoptysis (bleeding from lungs)
- Untreated pneumothorax (untreated collapsed lung)
- Known or suspected tympanic membrane rupture or other middle ear pathology

## ⚠ WARNINGS

Use of the device at excessive pressures may have adverse effects. Expiratory pressures above 20 cm H<sub>2</sub>O in patients sensitive to increased transpulmonary pressure may develop one or more of the adverse side effects listed below. Expert clinical judgment should be exercised in the selection of the appropriate setting for each individual patient. Failure to match the appropriate resistance setting on the Resistance Indicator with the patient's expiratory flow may result in failure to achieve therapeutic objectives of oscillatory PEP therapy or one or more adverse side effects listed below.

Adverse reactions may include:

- Increased work of breathing that may lead to hypoventilation and hypercarbia
- Increased cranial pressure
- Cardiovascular compromise
  - Myocardial ischemia
  - Decreased venous return
- Air swallowing with increased likelihood of vomiting and aspiration
- Claustrophobia
- Pulmonary barotraumas

## USE INSTRUCTIONS

Before use, ensure these instructions have been read and are kept available at all times. If you have questions about the performance or use of this product, please contact your healthcare professional prior to use. For device descriptions see page 1, figure 1.

**⚠ Warning:** Do not leave the device unattended with children. Contains small parts that can cause choking.

## DEVICE SET UP

The **Aerobika**\* device resistance setting should be set up by a healthcare professional using the following steps:

**1** Start with the Resistance Indicator at the middle setting. (page 1, figure 3)

**2** With the patient in a comfortable seated position, have them inhale through the device taking a deeper

breath than normal, but not to total lung capacity, then exhale actively but not forcefully through the device.

**3** Resistance should be set so exhalation lasts for 3–4 times longer than inhalation. Resistance can be increased by moving the indicator to the left. To reduce resistance, move the indicator to the right.

**Note:** Selection of the proper resistance setting will produce the desired inspiratory:expiratory flow ratio of 1:3 or 1:4 for 10–20 minutes without excess fatigue.

**4** Patients should be reevaluated periodically or with any change in status. As airways become less congested, more resistance will be needed to maintain inspiratory:expiratory ratios.

## DEVICE USE

Treatments should be conducted twice daily or as directed by your healthcare professional. Although the **Aerobika**\* device is orientation independent, it is recommended that treatments be done in a comfortably seated position. (page 2, figure 4)

**1** Carefully examine the device.

- If there is any visible dust or other debris inside, then clean the device. Replace immediately if it is damaged or has missing parts.
- Ensure the Resistance Indicator is set as prescribed by your healthcare professional.
- Gently press the two halves together to make sure the Attachment Tabs are engaged.

**2** Put the mouthpiece in mouth and close lips around it to ensure an effective seal. Make sure your hand does not block the exhalation path on the back of the device.

**3** Inhale through the device taking a deeper breath than normal, but do not totally fill your lungs. Hold your breath for 2–3 seconds before exhaling.

**4** Exhale actively, but not forcefully, through the device. Ideally, exhalation should last 3–4 times longer than inhalation. Keep your cheeks flat and firm to maximize treatment effectiveness. Ensure you maintain a good seal on the mouthpiece for the duration of the treatment.

**Note:** If you are unable to exhale through the device, check the resistance setting or try cleaning the device thoroughly.

**5** Continue taking deep breaths and long exhalations for 10–20 breaths, or as instructed by your healthcare professional. After at least 10 breaths, perform 2–3 'huff' coughs to clear your airways. Your healthcare professional will instruct you on the proper technique. For maximum efficacy, try to suppress coughing until the end of the treatment.

**6** Repeat steps 3–5 for 10–20 minutes or as prescribed by your healthcare professional. The entire procedure should be performed at least twice a day on a regular basis, and increased to 3–4 times per day if you are producing a lot of mucus.

The goal is to keep your lungs clear and ventilatory capacity up. Continue with treatments on a regular basis even if you are bringing up just a little mucus.

## USE WITH A NEBULIZER

The **Aerobika**<sup>®</sup> device may also be used with a small volume nebulizer with a 22 mm fitting. Your healthcare professional will advise which medication to use for combined treatments.

Medications which open the airways (eg. albuterol) or help to thin mucus (eg. hypertonic saline) would be good choices to use with the **Aerobika**<sup>®</sup> device because they help to remove or thin the mucus in your lungs.

**⚠ Caution:** Medications to stay in the lungs, like antibiotics, should be taken with the nebulizer alone after completing a treatment with the **Aerobika**<sup>®</sup> device.

**1** Ensure you have read and understood the instructions provided with the nebulizer prior to starting the treatment.

**2** Ensure the nebulizer is set-up correctly and medication has been added to the nebulizer cup. Remove the mouthpiece (or mask) from the nebulizer.

**3** Attach the nebulizer to the Nebulizer Port of the **Aerobika**<sup>®</sup> device, as shown in figure [2](#) (page 1).

**4** Switch on the compressed air source, and complete the combined treatment using the breathing technique outlined above.

### Notes:

- No adapter is required if used with the **AEROCLIPSE**<sup>®</sup> brand of breath actuated nebulizers.
- Ensure the nebulizer remains upright for proper nebulization.
- Always ensure the **Aerobika**<sup>®</sup> device mouthpiece is used, otherwise the device will not work properly. (page 1, figure [2](#))
- If used in combination with a nebulizer, the **Aerobika**<sup>®</sup> device should be cleaned after each use to remove any residue that may have deposited on exhalation.

## CLEANING AND DISINFECTING INSTRUCTIONS

Healthcare professionals should always use clinical judgment to select the appropriate cleaning and disinfecting method and frequency, based on patient needs.

If necessary, clean the **Aerobika**<sup>®</sup> device once daily.

**Note:** Proper and regular cleaning of the device is important for it to work correctly.

### 1 DISASSEMBLY

i. Press the Attachment Tabs on either side of the device. Holding the tabs in, gently pull up on the top case until the two parts separate.

ii. Remove the valve cartridge. Do not disassemble any part of the cartridge.

iii. To remove the mouthpiece, grip firmly and rotate while pulling away from the device.

### 2 CLEANING

Clean the **Aerobika**<sup>®</sup> device using one of the two following methods.

#### A Soap and Water

i. Wash the four parts in a solution of tap water and liquid dish detergent, allowing the parts to soak for 15 minutes. Agitate gently several times.

ii. Rinse with tap water.

iii. Shake the parts to remove any excess water.

#### B Dishwasher

i. Secure the four parts in a basket on the top rack.

ii. Wash in a normal wash cycle with dishwasher detergent and a rinse aid if available.

iii. Shake the parts to remove any excess water.

**Note:** Dishwashing with overly dirty dishes is not recommended.

### 3 REASSEMBLY

i. Ensure parts have air dried thoroughly.

ii. Insert the valve cartridge into the bottom case. If it does not slip easily into place, rotate 180° and try again.

iii. Connect the top and bottom cases until both Attachment Tabs “click” (this may occur simultaneously). Ensure the tabs are engaged by firmly pressing the two halves together.

iv. Reattach the mouthpiece.

### 4 DISINFECTION

If considered clinically necessary for the patient, healthcare professionals can instruct the patient to clean and disinfect the device following specific methods and frequencies (after each use).

Prior to disinfection, follow the disassembly and cleaning instructions but do not reassemble the device.

Disinfect the **Aerobika**<sup>®</sup> device using any one of the following methods.

#### A Electronic Steam Disinfector

i. Place the four parts in an electronic steam disinfector and follow the manufacturer’s instructions for use.

ii. Remove from the unit and allow the parts to air dry thoroughly before reassembly.

#### B Boiling

i. Place the four parts in a pot with boiling water and boil for 5 to 15 minutes.

**Note:** Place a small rack on the bottom of the pot to prevent plastic parts from coming in contact with the hot bottom.

ii. Remove from the water without touching the water and allow parts to air dry thoroughly before reassembly.

**Note:** Tap water can be high in minerals which may result in scale build-up in the device. Depending on the amount of time you boil the device, a more frequent vinegar soak may be required to reduce scale build-up in the device. Please see the chart below for the suggested vinegar soak frequency to keep your device performing well.

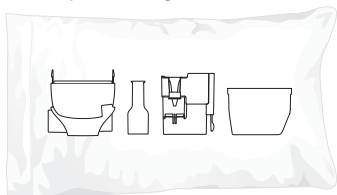
Boil Duration	Vinegar Soak Frequency
5 minutes	Monthly
5+ minutes	Every two weeks

Disassemble the device and soak in a solution of 1 cup vinegar and 1 cup water for 1 hour, then wash as normal.

### C Microwave Steam Bag

i. Place the four parts in a microwave steam bag (i.e. Quick-Clean<sup>†</sup> MicroSteam<sup>†</sup> bag) and follow the manufacturer's instructions for use.

**Note:** Position parts in bag as shown:

ii. Remove from bag and allow parts to air dry thoroughly before reassembly.

### D Isopropyl Alcohol

i. Soak the four parts in 70% isopropyl alcohol for 5 minutes.

ii. Rinse thoroughly with sterile water.

iii. Allow to air dry thoroughly before reassembly.

### E Hydrogen Peroxide

i. Soak the four parts in 3% hydrogen peroxide for 30 minutes.

ii. Rinse thoroughly with sterile water.

iii. Allow to air dry thoroughly before reassembly.

### F Bleach

i. Soak the four parts in a 1:50 bleach to water solution (approximately 1 tablespoon bleach in 3/4 cups water) for 3 minutes.

**Note:** You may substitute a respiratory equipment disinfectant (such as Milton<sup>†</sup>, Dodie<sup>†</sup>, Control III<sup>†</sup> or Cidex<sup>†</sup>) for the bleach solution. Follow the disinfectant instructions for use.

ii. Rinse thoroughly with sterile water.

iii. Allow to air dry thoroughly before reassembly.

### ① NOTE

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State.

### STORAGE

The device is fully dry and assembled, store in a clean plastic bag or container.

### RECYCLING

The **Aerobika**<sup>®</sup> OPEP device is recyclable. Recycling may be limited in your jurisdiction or local regulations may not permit it. Please consult with your community's recycling guidelines to determine availability of curbside recycling or recycling at a drop-off facility. Clean your device prior to recycling.

**Note:** The manometer gauge is not recyclable, please remove the manometer gauge prior to recycling (refer to figure [7]).

### DISPOSAL

The device can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective member countries.

### ADDITIONAL INFORMATION

Use Temperature Range	+15 °C to +40 °C
Storage Temperature Range	-40 °C to +60 °C

- Product is free of Bisphenol A, Phthalates, Latex, Lead and PVC.

## Manometer Accessory

### INTENDED USE

The manometer is intended to be used as a single patient use accessory to the **Aerobika**<sup>®</sup> Oscillating Positive Expiratory Pressure (OPEP) device as an indicator for patient expiratory pressure.

### CLINICAL BENEFIT

The clinical benefit to the patient is the ability to have a feedback while using the **Aerobika**<sup>®</sup> OPEP device. The manometer serves as an indicator for patient expiratory pressure. It is an optional accessory to the **Aerobika**<sup>®</sup> Oscillating Positive Expiratory Pressure (OPEP) device.

Performance Characteristic	10 L/min	30 L/min
Frequency (Hz)	6-8	14-18
Mean Pressure (cmH <sub>2</sub> O)	6-10	17-21
Amplitude (cmH <sub>2</sub> O)	10-16	16-20

### INSTRUCTIONS FOR USE

Please read the **Aerobika**<sup>®</sup> OPEP device Instructions for Use prior to using these instructions. An electronic version of these Instructions for Use is available for download at [www.trudellmed.com](http://www.trudellmed.com)

### SETTING UP YOUR MANOMETER WITH THE AEROBIKA<sup>®</sup> OPEP DEVICE

(See page 3, figure [6]. For device descriptions, see page 2, figure [5].)

Before use, ensure these instructions have been read and are kept available at all times. If you have questions about the performance or usability of this

device, please contact your healthcare professional prior to use.

**Note:** If there is any visible dust or other debris inside, then clean the device. Replace immediately if it is damaged or has missing parts.

**1** Remove the mouthpiece from the **Aerobika**<sup>®</sup> OPEP device.

**2** Insert the mouthpiece onto the end of the manometer adapter.

**3** Once the mouthpiece is inserted, ensure that the manometer gauge is securely attached in place and rotate so the numbers are facing upward.

**4** Insert the adapter onto the **Aerobika**<sup>®</sup> OPEP device in place of the mouthpiece as shown.

**Note:** Follow the proper method for using your **Aerobika**<sup>®</sup> OPEP device, contained in the Instructions for Use manual.

**5** As you exhale actively, the manometer gauge indicator will rise based on your exhalation pressure.

**6** For each exhalation, maintain the pressure level recommended by your healthcare professional.

### CAUTION

- The device scale is not for OPEP therapy. The correct pressure must be determined by a physician.

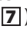
### WARNING

- Do not leave the device unattended with children. Contains small parts that can cause choking.

### NOTES

- SINGLE PATIENT USE
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State.

## CLEANING YOUR MANOMETER ADAPTER

The manometer adapter may be cleaned daily. (see page 3, figure )

**1** Remove the manometer adapter from the **Aerobika**<sup>®</sup> OPEP device.

**2** Remove the mouthpiece from the manometer adapter.

**3** Remove the manometer gauge from the manometer adapter and wipe down the exterior of the manometer gauge using a wipe (i.e., Accel PREvention, SoluPrep, Loris, Clorox Healthcare).

**NOTE:** The manometer gauge can be cleaned a maximum of 183 times (once daily for 6 months). At the end of 6 months, the sticker on the manometer gauge may begin to peel, this will not impact performance of the device.

**4** The mouthpiece and manometer adapter may be cleaned daily in soap and water, allowing parts to soak for 15 minutes. Agitate gently. Rinse parts in clean water.

### OR

Place the mouthpiece and manometer adapter in a dishwasher basket on the top rack. Wash in a normal

cycle. Dishwashing with overly dirty dishes is not recommended.

Shake to remove excess water and allow to air dry thoroughly before reassembly.

 **CAUTION:** The manometer gauge cannot be cleaned by soaking. It is a sealed unit.

## DISINFECTING YOUR MANOMETER ADAPTER

The mouthpiece and manometer adapter may be disinfected weekly using any of the following methods:

- Boil for 5 minutes.
- Microwave Steam Bag (according to manufacturer's instructions).
- Soak in 70% isopropyl alcohol for 5 minutes. Rinse well.
- Soak in 3% hydrogen peroxide for 30 minutes. Rinse well.

Allow to air dry thoroughly before reassembly.

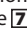
### CAUTION

- The manometer gauge cannot be disinfected with these methods.

### NOTES

- The manometer should be replaced after 6 months of use or immediately if damaged.
- This device has not been made with BPA (Bisphenol A) or natural rubber latex.
- Store the manometer with the **Aerobika**<sup>®</sup> OPEP device in a dry, dust-free place.

## RECYCLING

 The manometer gauge is not recyclable, please remove the manometer gauge prior to recycling (refer to figure )

## DISPOSAL

The manometer (adapter and gauge) can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective member countries.

## ADDITIONAL INFORMATION

Storage Conditions	-40° to 140°F (-40° to 60°C), ambient to 75% Relative Humidity (RH)
Operating Conditions	+59° to 113°F (+15° to 45°C), ambient to 85% RH
Measurement Error	0 – 30 cm H2O = ± 2 cm H2O 30 – 40 cm H2O = ± 3 cm H2O 40 – 60 cm H2O = ± 5 cm H2O

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Designed and developed in Canada /  
Conçu et mis au point au Canada

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**Oscillating Positive Expiratory Pressure  
Device with Manometer Accessory**

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**Manometer Accessory for use with AEROBIKA\*  
Oscillating Positive Expiratory Pressure Device**

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Scan or contact Trudell Medical  
International to obtain a glossary of  
the symbols used on packaging and  
instructions for use.

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