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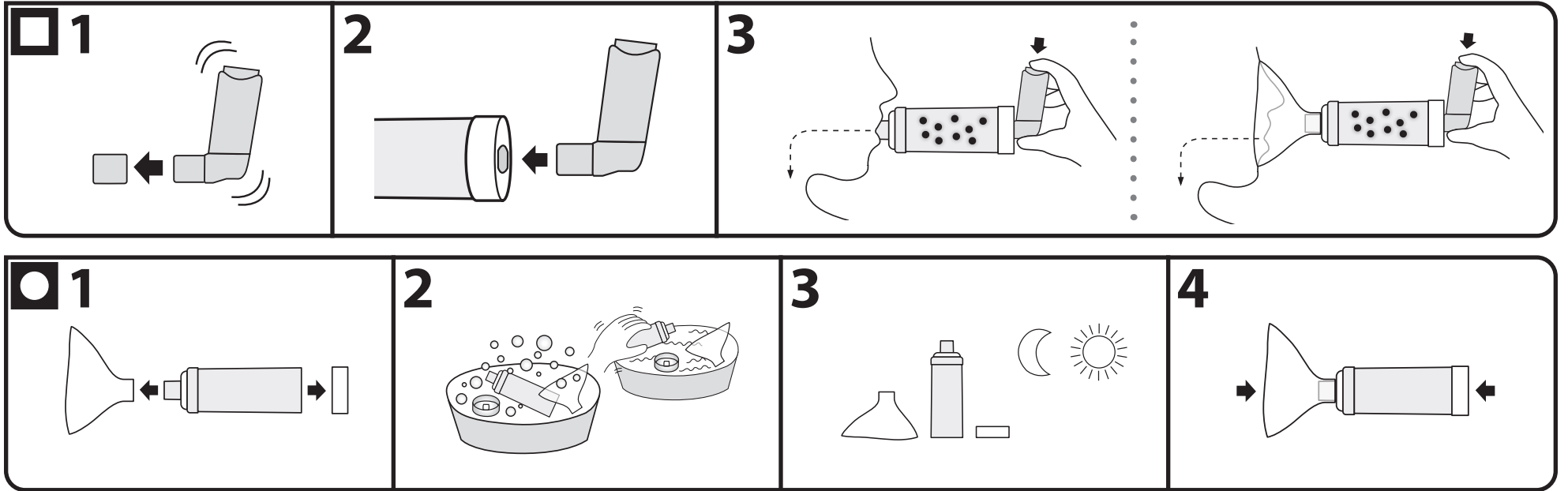
ES Grande  
FR Grand  
PL Duża  
PT Grande

EN

**Intended use:** This chamber is intended to be used by patients who are prescribed a metered dose inhaler, to deliver the aerosol medication from the inhaler to the lungs.

The clinical benefit of chambers is to improve the coordination of inhaling your medication. This is achieved by holding the medication, allowing you more time to inhale while filtering out the larger particles that would not reach your lungs. If you have any concerns regarding the use of the chamber or its delivery of your inhaler medication, please consult a healthcare professional.

### BEFORE EACH USE

This static-resistant chamber can be used directly out of package. Ensure these instructions and the instructions supplied with the inhaler have been read and are kept available at all times. Carefully look into the chamber and mask (if applicable). If there is any visible dust or other debris inside, then clean the chamber. Replace immediately if it is damaged or has missing parts.

### INSTRUCTIONS FOR USE

For preparation of the inhaler, follow the instructions supplied with the inhaler.

**1** Remove the cap from the inhaler. Shake the inhaler immediately before use as per the instructions supplied with it.

**2 MOUTHPIECE:** Insert the inhaler into the backpiece of the chamber. Put mouthpiece into mouth and close your lips around it to ensure an effective seal. **MASK:** Slide the mask onto the chamber. Insert the inhaler into the backpiece of the chamber. Apply mask to face and ensure an effective seal.

**3 MOUTHPIECE:** Exhale then press the inhaler once at the beginning of a slow inhalation. Inhale slowly and deeply through the chamber until a full breath has been taken. Hold your breath for 5-10 seconds before exhaling. **OR** Exhale and press the inhaler once at the beginning of a slow inhalation. Breathe in and out through the chamber for 2-3 breaths keeping lips sealed around chamber mouthpiece. **MASK:** Exhale then press the inhaler once at the beginning of a slow inhalation. Hold mask in place and breathe in and out through the chamber for 5-6 breaths.

### Notes

• Administer one (1) puff at a time. Follow instructions supplied with the inhaler on how long to wait before repeating.

• Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State.

### Cautions

• Do not use with Dry Powder Inhalers or Nebulizers. This chamber has not been designed to deliver these medications so you may not get the proper dose.

• Failure to follow instructions may affect medication delivery or cause inhalation issues.

### CLEANING

This chamber should be cleaned weekly.

**1** Remove the backpiece and the mask (if applicable).

**2** Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently. Rinse parts in clean water.

**3** Shake out excess water and allow to air dry in a vertical position. Ensure parts are dry before reassembly.

**4** To reassemble, press firmly to attach the backpiece. Apply mask if applicable.

### Notes

• The chamber should be replaced after 12 months of use.

• The chamber is free of Bisphenol A, Phthalates, Latex, Lead and PVC.

• To clean the inhaler, follow instructions supplied with the inhaler.

• The chamber can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective member countries.

• Store in a clean dry place when not in use.

### Cautions

Do not boil or sterilize. Product may be permanently damaged.

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