0-18 mon

1-5 yrs

5+ yrs

REF 113536

REF 113537

REF 113557

Emergo Europe
Westervoortsedijk 60,
6827 AT Arnhem,
The Netherlands

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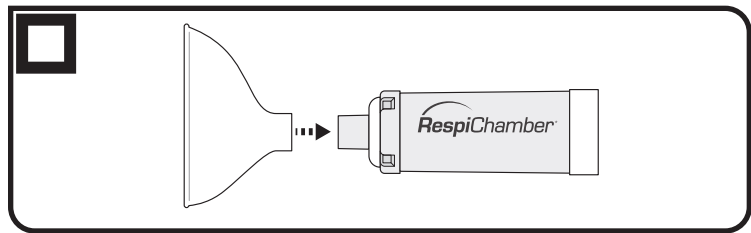
EN
INTENDED USE

This mask is intended for use with the **RespiChamber®** Valved Holding Chamber. The clinical benefit of masks is to enable treatment for patients who cannot use a mouthpiece. The mask enables delivery of aerosol medication with low inhalation resistance from the chamber. If you have concerns regarding the use of the mask, please speak with a healthcare professional.

BEFORE EACH USE

Ensure these instructions have been read and are kept available at all times. Carefully look into the mask. If there is any visible dust or other debris inside, then clean the mask. Replace immediately if it is damaged.

INSTRUCTIONS FOR USE

Fit the correct size of facemask to the **RespiChamber®** device as shown.

CLEANING

Clean and replace the mask the same as the **RespiChamber®** device as noted in those instructions.

Notes

- The mask can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective member countries.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State

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