
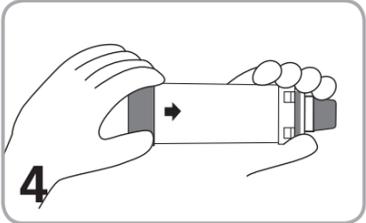
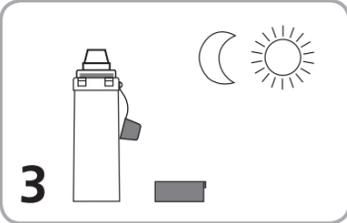
AeroChamber Plus Z STAT*

Anti-Static Valved Holding Chamber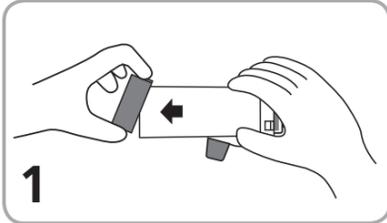

EN

CLEANING INSTRUCTIONS

This chamber can be used right out-of-package and then cleaned weekly.

1 Remove the backpiece only. Do not tamper with valve during cleaning or disassemble the product beyond what is recommended or damage may result. For mask product, do not remove mask.

2 Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently. Rinse parts in clean water. **OR** Place parts in top rack of dishwasher. Ensure product is securely placed face up. Run the dishwasher on a normal or light cycle and avoid heated dry.

3 Shake out excess water from the parts and allow to air dry in a vertical position. Ensure parts are dry before reassembly.

4 To reassemble, press firmly to attach the backpiece. For mouthpiece models, the protective cap should always be placed on the mouthpiece when the product is not in use.

ⓘ Notes

- Do not boil or sterilize.
- Product should be replaced after 12 months of use.
- This product contains no latex.
- Single Patient Use.
- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Dishwashing with overly dirty dishes is not recommended.
- If cleaning in a dishwasher use a rinse aid.

⚠ Cautions

- Product may be permanently damaged if boiled, sterilized or cleaned in a dishwasher at a temperature above 70°C.

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