


# Designed with patients in mind


With breath actuated delivery, aerosol is **only** produced when you breathe in. This means medication is not wasted between breaths or if you need to take a break during your treatment.

Medication is delivered **deep into the lungs** where it's most needed.<sup>1</sup>

Breath Actuated Delivery

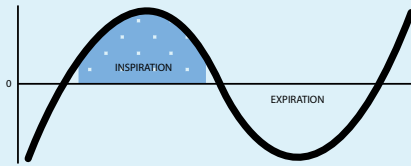

Conventional Continuous Nebulizers

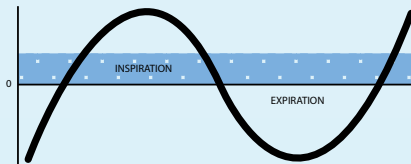

- **Green feedback button** moves down as you inhale to encourage slow, deep breath
- **Mode selector** to choose either breath actuated or continuous mode
- **Mouthpiece** can be removed and replaced with a mask
- **Exhalation valve** directs exhaled breath away from your face and eyes

Pair with an **Ombra**® Table Top Compressor for a complete delivery system at home.



Can be combined with the **Aerobika**® OPEP device to reduce overall treatment time. Speak to your healthcare provider for more information.



OPEP = oscillating positive expiratory pressure

**AeroEclipse™ XL**


BAN™ Nebulizer

**OMBRA**®


Compressors

## HOW TO USE


Nebulizer Top

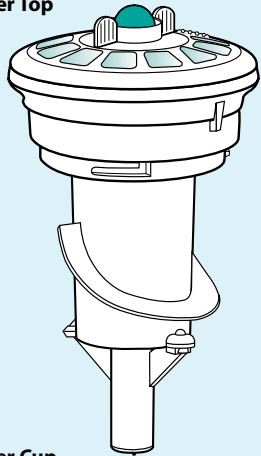

Nebulizer Cup

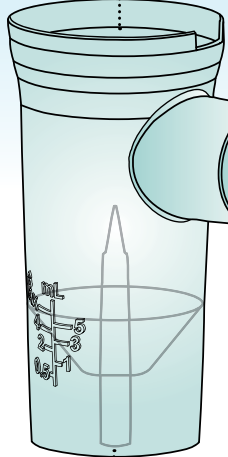

EZ Twist Tubing

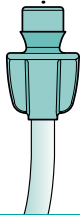

- 1 Place your compressor on a clean, flat surface.
- 2 Inspect your nebulizer and ensure the green exhalation valve on the mouthpiece is facing down. Unscrew and remove the nebulizer top. Carefully place prescribed medication into the nebulizer cup and replace the nebulizer top. Keep the nebulizer in an upright position to prevent spills. The nebulizer may be placed in the cup holder on the compressor.
- 3 Securely connect the tubing to the bottom of the nebulizer and to the air outlet on your compressor.
- 4 In a comfortably seated position, switch on your compressor. Place the mouthpiece in your mouth, ensuring your lips are sealed around the edge but aren't covering the green exhalation valve. If you are using a mask, place securely over the nose and mouth, ensuring a good seal.
- 5 Breathe in slowly and deeply, then exhale normally through the device. Continue taking slow, deep breaths until the nebulizer begins to make a 'sputter' noise. Switch off your compressor when your treatment is complete.

(Refer to the [instructions for use](#) for detailed guidance.)

Mouthpiece

Air Outlet

Nebulizer Cup Holder

On/Off Switch

*Ombra*® Table Top Compressor shown


## HOW TO CLEAN



**WASH**  
in soapy water



**RINSE**  
in clean water

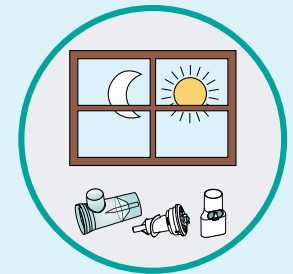

**AIR DRY**  
thoroughly

Notes:

---



---



---




---



---

**DISHWASHER SAFE**  
top rack in basket

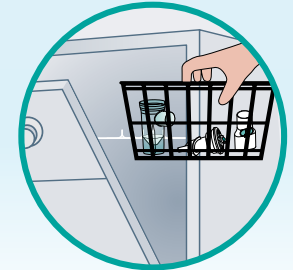

The *AeroEclipse*® *XL BAN*™ Nebulizer is a single-patient use device, reusable for 6 months.

## HOW TO DISINFECT

### BOILING:

The nebulizer (excluding tubing) may be boiled in water for 5-15 minutes†

### ELECTRONIC STEAM DISINFECTOR:

The nebulizer (excluding tubing) may be disinfected using an electronic steam sterilizer.

Refer to the [instructions for use](#) for additional information.

† Meets US Cystic Fibrosis Foundation guidelines for cleaning and disinfection<sup>2</sup>

### MICROWAVE STEAM BAG:

The nebulizer (excluding tubing) may be disinfected in the microwave in a steam cleaning bag.

### 70% ISOPROPYL ALCOHOL:

The nebulizer (excluding tubing) may be soaked in 70% isopropyl alcohol for 5 minutes‡

[aeroclipse.com](http://aeroclipse.com)

Manufactured by:



**TRUDELL MEDICAL**  
INTERNATIONAL

+1-866-510-0004  
info@trudellmed.com [trudellmed.com](http://trudellmed.com)



Designed and  
Developed in Canada  
Manufactured in Canada  
with Canadian and imported parts.

