

Go with
confidence



2 in 1 Chamber
+ Protective Case



Compact & Portable

Delivers inhaler medication better
than the inhaler alone^{1,2}

If you can **taste** your inhaler medication, it's likely not getting where it needs to go.


More medication is **delivered** to the lungs^{1,2}

Aero2go[™]
Chamber

Better2gether



Inhaler alone

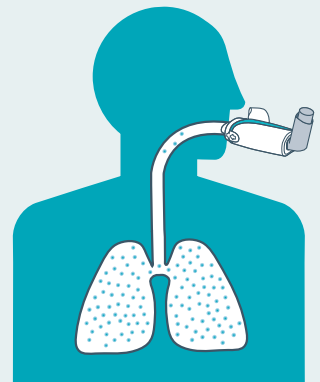


Inhaler with
AeroChamber2go[™] Chamber

Designed for use *on-the-go*



2 in 1 Chamber + Protective Case

Your inhaler can be stored inside the chamber.

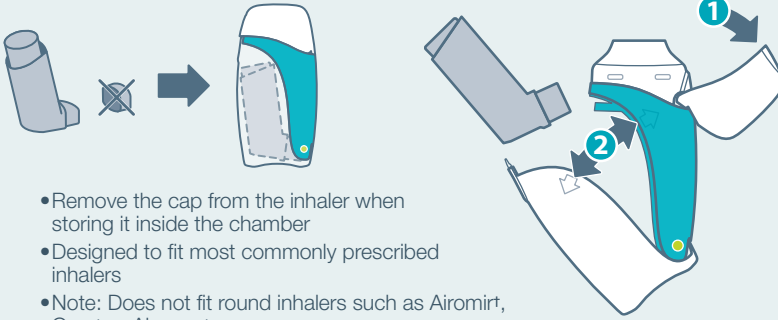

- Remove the cap from the inhaler when storing it inside the chamber
- Designed to fit most commonly prescribed inhalers
- Note: Does not fit round inhalers such as Airomirt, Qvar or Alvescot

Easy to clean + all one piece

The chamber can be used directly out of the package and should be cleaned weekly.



www.aerochamber2go.com

for instructional videos




Shake

inhaler immediately before use (remove cap)



Insert

inhaler into back of the chamber

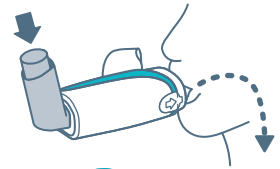

Press

inhaler as you start to breathe in slowly and deeply

Chambers for use at home and *on-the-go*



www.myAeroChamber.com

JOIN to receive exclusive offers and **FREE** giveaways!



TRUDELL MEDICAL INTERNATIONAL



Designed and Developed in Canada with the help of people like you

PART OF THE **AeroChamber®** FAMILY OF CHAMBERS