
2 in 1
Chamber +
Protective Case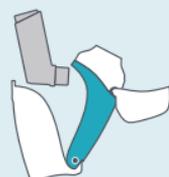

Aero2go
Chamber

Anti-Static Valved Holding Chamber

www.aerochamber2go.com

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TRUDELL MEDICAL
INTERNATIONAL

725 Baransway Drive, London,
Ontario, Canada N5V 5G4
T +1-519-455-7060
F +1-519-455-6478
info@trudellmed.com

INTENDED USE

The **AeroChamber2go™** chamber is intended to be used with most pressurized metered dose inhalers to deliver aerosolized medication to the lungs.

If you have any questions, please contact your healthcare provider.

HOW IT WORKS


When you press the inhaler, the chamber holds the medication suspended. This allows you more time to inhale and helps to deliver the medication to the lungs, where it is needed.

USING YOUR CHAMBER


Shake

inhaler immediately
before use
(remove cap)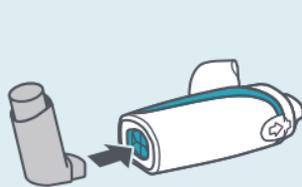

Insert

inhaler into back
of the chamber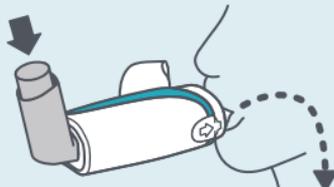

Press

inhaler as you start
to breathe in **slowly**
and **deeply**

Notes

- Ensure these instructions and the instructions supplied with the inhaler have been read and are kept available at all times.
- Take 1 puff at a time.
- The chamber should be replaced after 12 months of use.

Warnings

- Carefully look into the chamber. If there is any visible dust or other debris inside, then clean the chamber.
- Replace immediately if the chamber is damaged or has missing parts.

Cautions


- Do not use with Soft Mist Inhalers (i.e. Respimat[†]), Dry Powder Inhalers or Nebulizers. The chamber has not been designed to deliver these medications so you may not get the proper dose.
- Failure to follow instructions may affect medication delivery or cause inhalation issues.

Tips

- For best results, hold your breath for as long as it feels comfortable before exhaling. This improves the effectiveness of the medicine.
- If you are not able to hold your breath, you can simply press the inhaler and breathe normally in and out 2 times through the chamber to inhale your medicine.

2 IN 1 CHAMBER + PROTECTIVE CASE FOR YOUR INHALER

Your inhaler can be stored inside the chamber.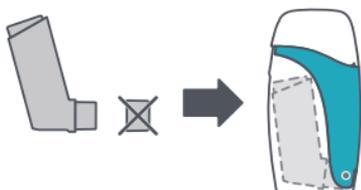


- Remove the cap from the inhaler when storing it inside the chamber.
- Chamber is enclosed on both ends helping to prevent debris from entering.
- Designed to fit most commonly prescribed inhalers. (Note: inhaler is sold separately)

Caution

- Not designed for use with round mouthpiece inhalers such as Airomirt[†], Qvar[†] or Alvesco[†]. Use may cause premature wear or damage to device.

To open chamber: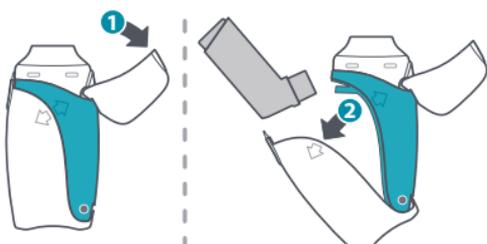


Reverse order to close. (close 2 then 1)

Warning


- Ensure you (and if applicable your child) can open the chamber before storing your inhaler inside.
- The chamber is not intended for use by children less than 5 years of age.

CLEANING YOUR CHAMBER

The chamber can be used directly out of the package and should be cleaned weekly.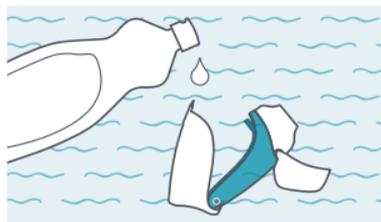

OR

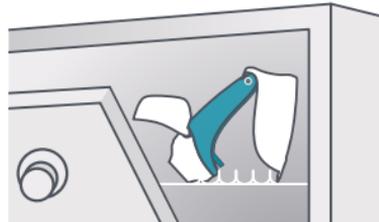1 Lift the cap and open the chamber to remove the inhaler.

2 Wash the chamber in warm, clean water with liquid dish detergent, agitating gently. Rinse in clean water.

OR Place on top rack of dishwasher as shown. Run the dishwasher on a normal or light cycle. Do not heat dry.

3 Shake out excess water and allow to air dry. If you require the chamber immediately, dry gently with a clean dish towel.

Notes

- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a clean soft cloth.
- Do not boil or sterilize.
- Dishwashing with overly dirty dishes is not recommended.
- If cleaning in a dishwasher, use a rinse aid.
- For cleaning the inhaler, follow the instructions supplied with the inhaler.

Caution

- Product may be permanently damaged if boiled, sterilized or cleaned in a dishwasher at a temperature above 70°C.

Scan for
instructional videos

Designed and Developed
in Canada with the help
of people like you

Chambers should be
replaced every 12 months
for optimum performance.

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